
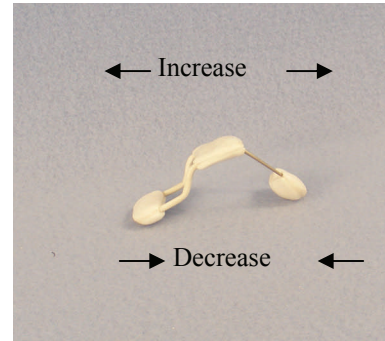


FW07 Instructions

INDICATIONS—Increase P.I.P. joint extension

APPLICATION—With curved lateral wires proximally slide finger into splint. Position distal finger band proximal to DIP Joint and proximal band at Metacarpal head.

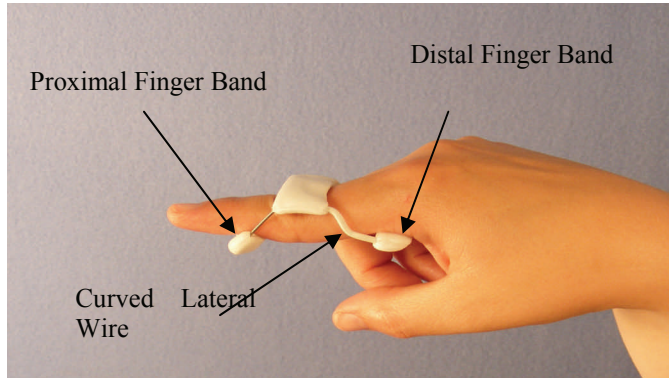

ADJUST EXTENSION FORCE—With splint removed:

To Increase—Bend ends of splint apart

To Decrease—Bend ends of splint toward each other

TO BE FIT BY A QUALIFIED HEALTHCARE PROFESSIONAL

Qualified Healthcare Professional to provide use and wearing instructions/ guidelines.

PRECAUTIONS—Immediately contact your healthcare professional if you are experiencing pain, swelling, numbness, skin discoloration, or have any questions or concerns.

CLEANING—Hand wash with damp towel, cold water and mild soap. Thoroughly rinse and air dry completely before reapplying.

Contraindicated for fixed or boney contractures

CAUTION: Federal Law restricts this device to sale by or on the order of a physician



Single patient use